

**Instructions for Use**  
**FASENRA PEN™ (fas-en-rah)**  
**(benralizumab)**  
**for Subcutaneous Injection**  
**Single-dose Autoinjector**

Before using your FASENRA PEN, your healthcare provider should show you or your caregiver how to use it the right way. In people aged 6 to 11 years, FASENRA should only be given by a caregiver or healthcare provider.

**Read this Instructions for Use before you start using your FASENRA PEN and each time you get a refill.** There may be new information. This information does not take the place of talking to your healthcare provider about your medical condition or your treatment.

If you or your caregiver have any questions, talk to your healthcare provider.

**Important information:**

- **Store FASENRA in a refrigerator between 36°F to 46°F (2°C to 8°C) in its original carton until you are ready to use it.**
- FASENRA may be kept at room temperature between 68°F to 77°F (20°C to 25°C) for a maximum of 14 days.
- Once removed from the refrigerator and brought to room temperature, FASENRA must be used within 14 days or thrown away (disposed of).

**Do not** use your FASENRA PEN if:

- it has been frozen
- it has been dropped or damaged
- the security seal on the carton has been broken
- the expiration date (EXP) has passed

**Do not:**

- shake your FASENRA PEN
- share or reuse your FASENRA PEN
- expose your FASENRA PEN to heat


If any of these happen, throw away the FASENRA PEN in a puncture-resistant sharps disposal container and use a new FASENRA PEN.

Each FASENRA PEN contains 1 dose of FASENRA that is for one time use only.

**Keep FASENRA and all medicines out of the sight and reach of children.**


**Your FASENRA PEN**

**Do not** remove the cap until you have reached Step 6 of these instructions and are ready to inject FASENRA.

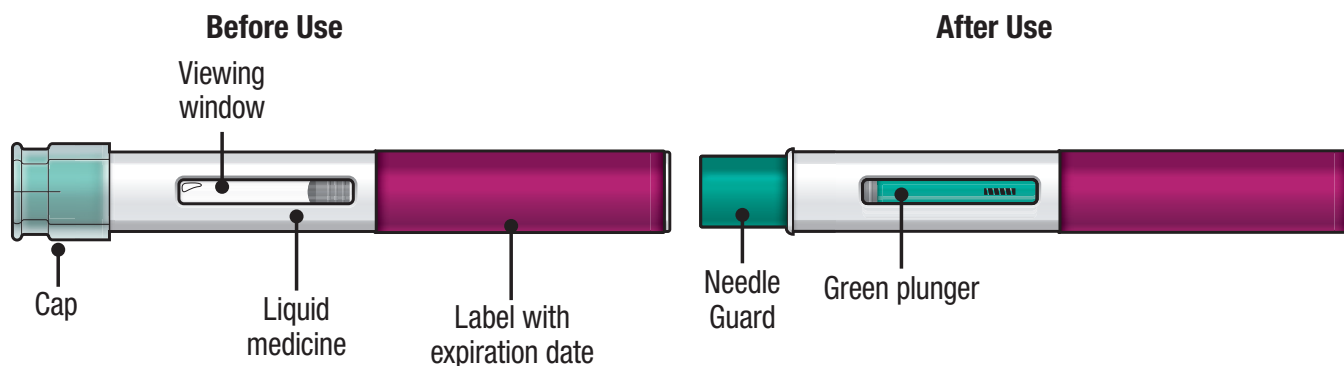

**Step 1 – Gather supplies**

- 1 FASENRA PEN from the refrigerator
- 1 alcohol wipe
- 1 cotton ball or gauze
- 1 bandage
- 1 puncture-resistant sharps disposal container. See Step 10 for instructions on how to throw away (dispose of) the used FASENRA PEN safely.

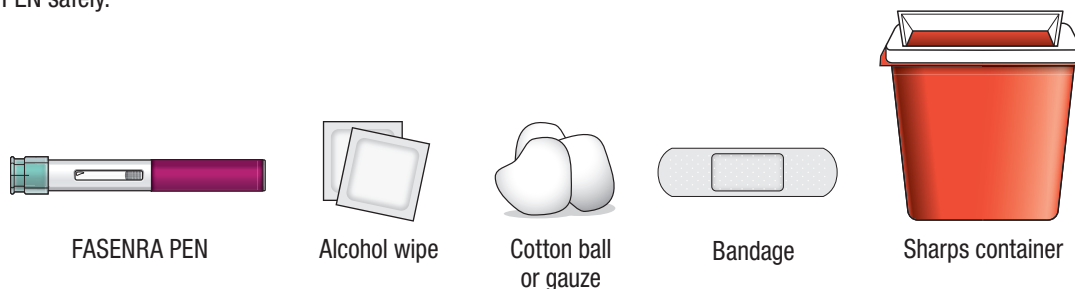

## Step 2 – Prepare to use your FASENRA PEN

**Check the expiration date (EXP).** Do not use if the expiration date has passed.

**Let FASENRA warm up at room temperature** between 68°F to 77°F (20°C to 25°C) **for about 30 minutes before giving the injection.**


**Do not** warm the FASENRA PEN in any other way. For example, do not warm it in a microwave or hot water, or put it near other heat sources.

Use FASENRA within 14 days of removing from the refrigerator. After 14 days, throw away the FASENRA PEN.

**Do not** remove the cap until you have reached Step 6.



## Step 3 – Check the liquid

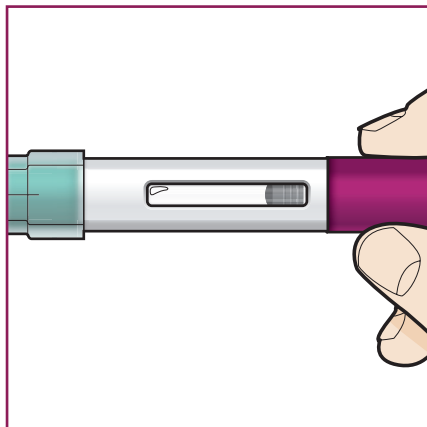


**Look at the liquid in the FASENRA PEN through the viewing window.** The liquid should be clear and colorless to slightly yellow. It may contain small white particles.

**Do not** inject FASENRA if the liquid is cloudy, discolored, or contains large particles.

You may see small air bubbles in the liquid. This is normal. You do not need to do anything about it.

## Step 4 – Choose the injection site

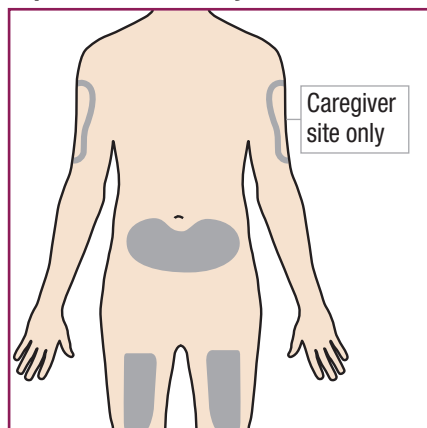

If you are giving yourself the injection, the **recommended injection site** is the front of your thigh or the lower part of your stomach (abdomen).


A caregiver may inject you in the upper-arm, thigh, or abdomen. **Do not** try to inject yourself in the arm.

For each injection, choose a different site that is at least 1-inch (3-cm) away from where you last injected.

**Do not** inject:

- into the 2-inch (5-cm) area around your bellybutton
- where the skin is tender, bruised, scaly or hard
- into scars or damaged skin
- through clothing

## Step 5 – Clean the injection site

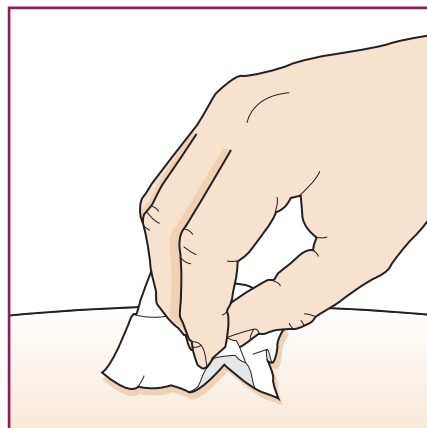


Wash your hands well with soap and water.

Clean the injection site with an alcohol wipe in a circular motion.  
Let it air dry.

**Do not** touch the cleaned area before injecting.

**Do not** fan or blow on the cleaned area.

### Step 6 – Pull off the cap

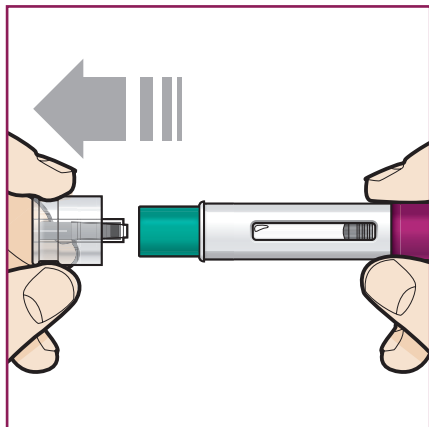

Hold the FASENRA PEN with 1 hand. Carefully pull the cap straight off with your other hand.

Put the cap aside to throw away later.

The green needle guard is now exposed. It is there to prevent you from touching the needle.

**Do not** try to touch the needle or push on the needle guard with your finger.

**Do not** try to put the cap back on the FASENRA PEN. You could cause the injection to happen too soon or damage the needle.

Complete the following steps right away after removing the cap.


### Step 7 – Inject FASENRA

Follow your healthcare providers instructions on how to inject. You can either gently pinch at the injection site or give the injection without pinching the skin.

Inject FASENRA by following the steps in figures **a**, **b**, **c**, and **d**.


Hold the FASENRA PEN in place for the entire injection.

**Do not** change the position of the FASENRA PEN after the injection has started.

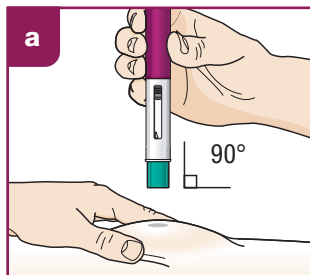


#### Position the FASENRA PEN at the injection site.

Place the needle guard of the FASENRA PEN flat against your skin (90 degree angle). Make sure you can see the viewing window.

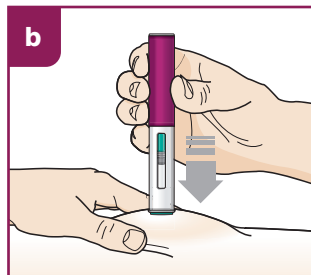


#### Press down firmly.

You will hear a click. A 'click' tells you the injection has started. The green plunger will move down in the viewing window during the injection.

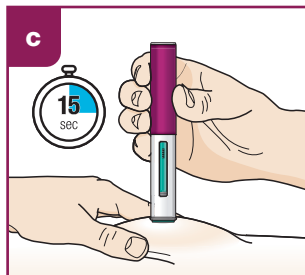

#### Hold down firmly for 15 seconds.


You will hear a second 'click'. The second click tells you the injection has finished. The green plunger will fill the viewing window.

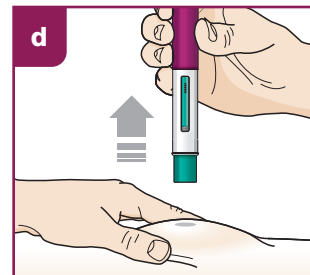

#### Lift the FASENRA PEN straight up.


The needle guard will slide down and lock into place over the needle.

### Step 8 – Check the viewing window

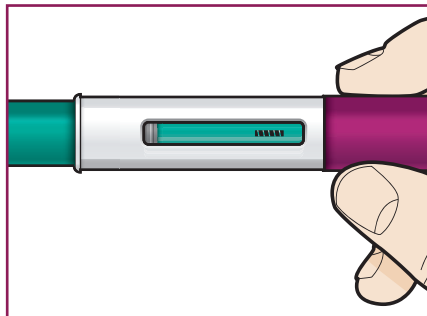


Check the viewing window to make sure all the liquid has been injected.

If the green plunger does not fill the viewing window, you may not have received the full dose. If this happens or if you have any other concerns, call your healthcare provider.

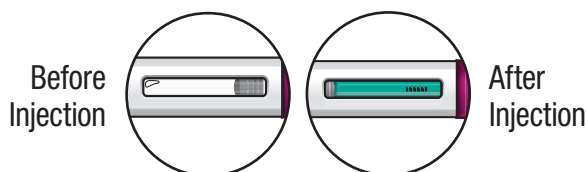

### Step 9 – Check the injection site

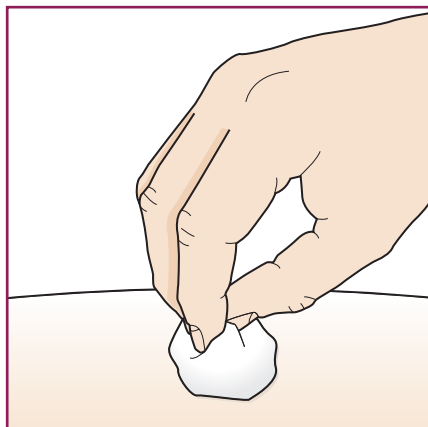


There may be a small amount of blood or liquid where you injected. This is normal. Gently hold pressure over your skin with a cotton ball or gauze until the bleeding stops.

**Do not** rub the injection site.

If needed, cover the injection site with a small bandage.

### Step 10 – Dispose of the used FASENRA PEN safely

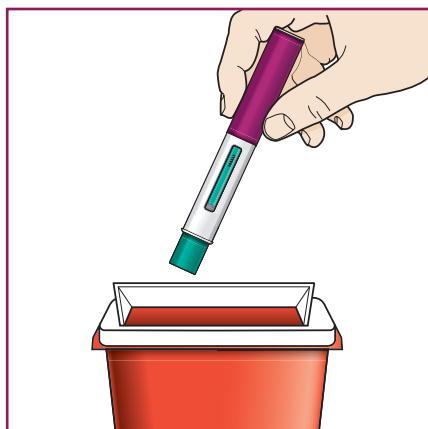

- Each FASENRA PEN contains a single dose of FASENRA and **cannot be reused**.
- Put your used FASENRA PEN in a FDA-cleared **sharps disposal container** right away after use.

**Do not** throw away the FASENRA PEN in your household trash.

Throw away the cap and other used supplies in your household trash.

### Disposal guidelines

If you do not have a FDA-cleared sharps disposal container, you may use a household container that is:

- made of a heavy-duty plastic,
- can be closed with a tight-fitting puncture-resistant lid, without sharps being able to come out,
- upright and stable during use,
- leak-resistant, and
- properly labeled to warn of hazardous waste inside the container.

When your sharps disposal container is almost full, you will need to follow your community guidelines for the right way to dispose of your sharps disposal container. There may be state or local laws about how you should throw away used needles and syringes. For more information about safe sharps disposal, and for specific information about sharps disposal in the area that you live in, go to the FDA's website at: <http://www.fda.gov/safesharpsdisposal>.

**Do not** dispose of your used sharps disposal container in your household trash unless your community guidelines permit this.

**Do not** recycle your used sharps disposal container.

For more information go to [www.FasenraPen.com](http://www.FasenraPen.com) or call 1-800-236-9933. If you still have questions, call your healthcare provider.

Manufactured by: AstraZeneca AB, Södertälje, Sweden SE-15185

U.S. License Number 2059

Distributed by: AstraZeneca Pharmaceuticals LP, Wilmington, DE 19850

FASENRA is a trademark of the AstraZeneca group of companies.

©AstraZeneca 2024